## HEINE® HR/HRP/ HR-C Binocular Loupes

CE  
HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 - 82211 Herrsching - Germany  
Tel. +49(0)8152/38-0  
Fax +49(0)8152/38-202  
E-Mail: info@heine.com • www.heine.com  
med 2606 1/9.12

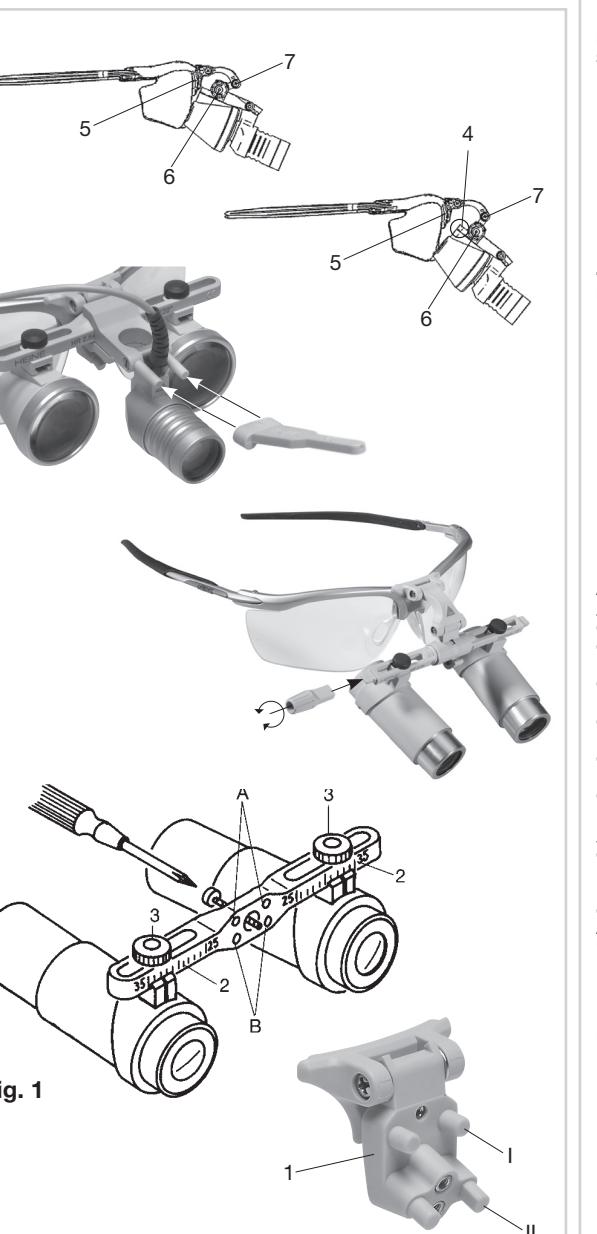
Fig. 1

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DEUTSCH

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B-I: horizontal    B-II: geneigt    A-II: as in B-II, but mounted 5 mm lower  
Your loupe is fitted to one of the following carrier systems. The swivel attachment can be fitted to both the S-Frame and the Combi frame.

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#### MD 1000 F.O. and 3S LED HeadLight: Mount the HEINE binocular loupes by means of the loupes adapter or HEINE S-Guard® to the Headlight. To do so, please consult the instructions supplied with the HEINE S-Guard®.

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## Instructions

ENGLISH

Before using your HEINE® HR/HRP/HR-C Binocular Loupes read these instructions carefully and keep them in a safe place for future reference.

### Safety information

**Range of applications:** HEINE® HR Loupes are designed only for optical magnification during examinations and operations.  
**Safety in use:** Do not look directly at the sun or into a powerful light source with the loupes to avoid the possibility of eye damage. The angle of convergence of the eyepieces was set by the manufacturer (4). Do not alter this setting. This may only be adjusted by HEINE at the factory.

### Assembly and use

#### How to fit HEINE binocular loupes with HEINE® i-View to the HEINE S-Frame®:

The HEINE® i-View allows you to select the angle of view of the loups individually. This is done by means of a triple link system (5, 6, 7).

#### Swiveling the loups:

The HEINE® i-View can be swivelled by hand as shown above. To do so, attach a steri-lever to one end of the optics mount. The loups can be moved out of the line of sight by rotating the lever. This allows you to view the object without magnification and avoids the need to re-adjust the light.

#### How to fit HEINE binocular loupes with HEINE® i-View to the Professional L Headband with S-Guard splash protection

Unscrew the fixing screw (8) and the S-Guard can be moved vertically and horizontally. Fix the loups with the fixing screw (8) as shown in Fig. in the desired position.

#### How to attach HEINE binocular loupes without HEINE® i-View to the HEINE S-Frame®:

HEINE® HR Loupes can be fixed to the frame or other carrier system in three different positions:

B-I: horizontal    B-II: angled slightly down    A-II: as in B-II, but mounted 5 mm lower

Your loupe is fitted to one of the following carrier systems. The swivel attachment can be fitted to both the S-Frame and the Combi frame.

#### Professional Headband: Circumference and height can be adjusted by means of the two controls.

#### MD 1000 F.O. and 3S LED HeadLight: Attach the HEINE binocular loupes by means of the loupes adapter or HEINE S-Guard® to the Headlight. To do so, please consult the instructions supplied with the HEINE S-Guard®.

#### Combi Frame: Adjust the length of the temples to suit your needs.

### Adjustment

The HEINE Loupe should be adjusted very carefully to assure the best possible view.

#### To begin setting up proceed as follows:

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#### The loupe is correctly adjusted when you see a round, shadow-free image.

#### In the correct position the loupe is held firmly in place by the swivel function.

- Fix the eyepieces in place by re-tightening the screws (3) (Fig.1).
- When the **swivel function** of the HEINE® i-View (Z, Fig.2) is used the position of the joints X and Y (Fig.2) should remain unchanged. If joints X and Y are unintentionally changed when the i-View is swivelled, tighten the screws X and Y (Fig.3) with the screwdriver supplied with the instrument. Correct tightening will prevent any unintentional adjustment.

### Accessories

#### HEINE binocular loupes without HEINE® i-View:

- The angled adaptor is a screw-fit to the lowest setting on the mount and allows the eye-pieces to be angled downwards. This allows the operator to keep a more level head position even when looking down to operate close to his own body (does not apply to the MD 1000 F.O.).
- The washable lens protectors are push-fit to the eyepieces.
- The optional lights for binocular loupes (LED LoupeLight) are described in the appropriate instructions.
- Only for **HR-Binokularlupen**.
- The close-up lenses shorten the working distance from 340 to 250 mm and are a simple push-on fit.

### Cleaning and Maintenance

#### Manual Cleaning

- o Before cleaning, remove the lens protectors.
- o Use a microfiber cleaning cloth when removing fingerprints from the optics.
- o Use a damp cloth when cleaning the housing and the optics (water, cleaning solutions with a neutral or mildly alkaline pH).
- o The loups are water resistant and can be washed under running water.
- o If residues/dirt must be removed from the optics and/or the housing, use a soft brush.
- o Warning: Hard and/or metallic brushes and/or abrasive treatment mediums lead to scratches and therefore deterioration of the optics.
- o Disinfection by means of spraying or wiping is permitted. Apply cleaning solutions that have been approved for medical devices made from plastic and glass.
- o Apply isopropyl alcohol (only up to a maximum of 70% concentration).
- o Warning: Immersion disinfection, machine cleaning/disinfection, ultrasound cleaning and sterilisation are not permitted.

#### Cleaning the loupe mount HEINE® i-View

- Use a micro-fiber cloth with a little alcohol or non-aggressive solution to clean the loupe mount.

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## Technical specification

The values for magnification and working distance marked on the loupes refer to normally-sighted or optically-corrected eyes ignoring accommodation. These values can vary depending on refractive error and accommodation. In general, if the examiner is short-sighted or tends to accommodate the actual working distance will be shorter. If long-sighted, it will be longer. If the working distance is shorter, the magnification will be stronger and vice versa.

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