**HEINE** QUALITY  
MADE IN GERMANY

⌘ HEINE S-Frame®

⌘ HEINE S-Guard®

**S-Frame**

**S-Guard**

**HEINE Optotechnik GmbH & Co. KG**  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)81 52/38-0  
Fax +49(0)81 52/38-202  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
med 2106    2014-01-10

**HEINE S-Frame®**

**HEINE S-Guard®**

<b>DEUTSCH</b>	<b>5</b>
<b>ENGLISH</b>	<b>8</b>
<b>FRANÇAIS</b>	<b>11</b>
<b>ESPAÑOL</b>	<b>14</b>
<b>ITALIANO</b>	<b>17</b>
<b>NEDERLANDS</b>	<b>20</b>
<b>PORTUGUÊS</b>	<b>23</b>

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 1

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 2

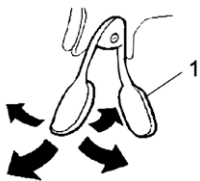
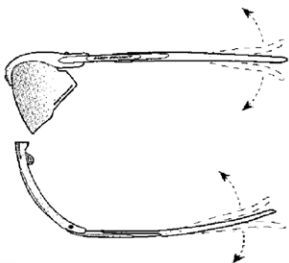
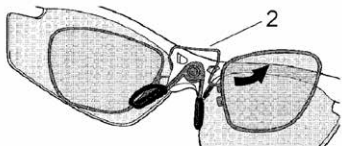
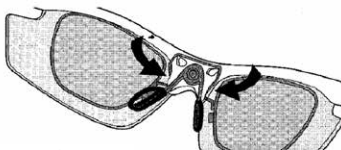
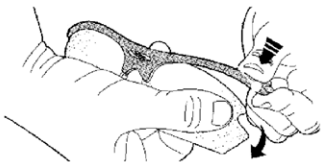
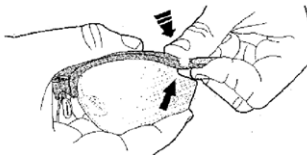
Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 4

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 5

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 3

**A****B****C****D****E****F****G**

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Please read and follow these instructions for use and keep them for future reference.

### General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

#### For U.S. only:

**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

### Warnings and Safety Information

**CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

**NOTE!** Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

### Intended Use

The HEINE S-Frame® and HEINE S-Guard® are designed for use exclusively with HEINE instruments and light sources.

### Operate

**Retaining Cord:** This stabilises your HEINE S-Frame® and prevents slipping. Push the ends of the cord firmly over the ends of the temples. Adjust the length of the cord to suit your needs.

**Adjusting the nose bridge:** (A) This can be bent into position to suit the anatomy of your nose. By bending the bridge supports you can vary the distance between the lenses and the eyes. This avoids contact between your eyelashes or cheekbones and the frame or corrective lenses.

**Changing the nose pad:** The nose pad is attached to the S-Frame by a screw. To change the nose pad, unscrew the screw and remove the old nose pad.

**Adjusting the temples:** (B) The temple ends can be bent into shape in any direction. Adjust them until they fit comfortably.

**Replacing a temple:** To replace a temple, simply unscrew the hinge screw, insert the new temple and screw in the hinge screw.

**Fitting optional corrective lenses:** Please consult an optician before having corrective lenses fitted. Please show the optician the attached "Instructions for specifying corrective lenses". To fit the corrective lenses (C), insert one side of the snap-in frame (2) into the groove in the S-Frame. (D) Next, insert the other side of the snap-in frame in the second groove.

To remove the snap-in frame with corrective lenses, reverse the above procedure, avoiding damage to the lenses.

**Changing protective lenses:** Hold the S-Frame as in (E) and bend the lens in the direction shown and pull the lens with an audible click out of the frame. To replace, follow the procedure shown in (F) and (G) until the lens clicks into place.

If the standard protective lenses are too close to your cheekbones, you can either fit the small protective lenses or adjust the nose bridge (A).

#### **Instructions for specifying corrective lenses:**

When specifying and fitting corrective lenses, be sure to take into account the following:

- The magnifying optics should be attached to the frame to suit your usual working posture.
- Prismatic errors should be corrected.
- The corrective lenses are mounted at an angle to the optical axis.

Check the correction by means of an eye test with the magnifying optics and the corrective lenses fitted.

The HEINE SIGMA® 150 eyepieces include 2 diopter lenses. This enables normalsighted, emmetropic patients to view the retinal image without effort.

The total spherical correction is usually max. +/- 5 dpt. Ask your optician or ophthalmologist!

The corrective lenses should be placed as closely as possible to the eyepieces of the magnifying optics.

#### **Description:**

a) As a frame with mounting bracket for HEINE Instruments:

- HEINE SIGMA® 150 Binocular indirect ophthalmoscope
- HEINE® HR Binocular loupes
- HEINE® HRP Binocular loupes
- HEINE® HR-C Binocular loupes

b) As eye-protection (Frame without temples), when mounted on the HEINE Professional L Headband fitted with the HEINE mounting bracket for HEINE Instruments:

- HEINE® MD 1000 Fiber Optic Headlight
- HEINE® 3S LED HeadLight
- HEINE® HR, HRP, HR-C Binocular loupes

#### **How to fit the HEINE SIGMA® 150 to the HEINE S-Frame®:**

Please follow the instructions supplied with the HEINE SIGMA® 150.

#### **How to fit HEINE Binocular loupes with the HEINE i-View to the HEINE S-Frame®:**

Screw the HEINE i-View with your HEINE Binocular loupe to the fixing plate (Fig. 3). Please follow the instructions for adjusting the HEINE Binocular loupes and HEINE LED LoupeLight according to the instructions supplied with these units.

#### **How to fit HEINE Binocular loupes (HR-C and HR 2,0x) without HEINE i-View to the HEINE S-Frame®:**

Attach the loupe adaptor to the S-Frame with the screw provided. Screw the binocular loupes to the swivel-mount (Fig.1). Please note that the loupes can be mounted at different heights and that the angle of view can be varied by fitting the HEINE dental adaptor (only HR-C and HR 2,0x). Please follow the instructions for adjusting the HEINE Binocular loupes and HEINE LED LoupeLight according to the instructions supplied with these units.

#### **Assembly of the S-Guard with other HEINE instruments:**

Screw your HEINE S-Guard® to the **Professional L Headband** as in Fig. 2.

Screw your HEINE S-Guard® to the **3S LED HeadLight** as Fig. 4.

Screw your HEINE S-Guard® to the **MD 1000 F.O. HeadLight** as Fig. 5.

#### **Hygienic Cleaning / Processing**

**General:** The plastic protective lenses have an anti-reflex coating. Clean them carefully to avoid mechanical or chemical damage.

**Manual Cleaning:** Clean the S-Frame and the S-Guard with the enclosed microfibre cloth. We recommend the HEINE cleaning spray C-000.32.542 as a cleaning agent.

**Disinfectant wipes/Spray disinfection:** Apply disinfectants that have been approved for medical devices made from synthetic material and glass. We recommend the application of isopropyl alcohol at maximum 70 % concentration.

**Warning:** Immersion disinfection, machine cleaning/disinfection, ultrasound cleaning and sterilisation are not permitted.

## Maintenance

This product does not require regular maintenance.

## Service

This product does not require regular service.

## Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

## Technical specification

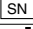
Weight: 25 g  
Protective lenses: Plastic, anti-reflex coated  
Frame: Nickel-free

## General Notes and Warnings

-  Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.
  - To avoid possible damage to the eyes, avoid looking directly at the sun or a bright light!
  - HEINE declines responsibility for any assembly defects caused by the optician.
  - To avoid possible damage to the frame, avoid frequent bending of the temples.
-  This product is a precise optical instrument. Please handle this product with care and store it in a clean place.
  - Use only HEINE original replacement parts and accessories.
-  This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

## Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

	The CE mark indicates that the product complies with the European medical device directive 93/42/EEC.
	Catalogue- or reference number
	Serial number
	Manufacturer
	Date of manufacture
	Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE)
	Temperature limits in °C for storage and transport
	Temperature limits in °F for storage and transport
	Humidity limitation for storage and transport
	Please read and follow the instructions for use and keep them for future reference (background color blue; foreground color white)
	Fragile, handle with care!
	Keep dry
	Interference may occur in the vicinity of equipment marked with the following symbol
	"Grüner Punkt" (country-specific)

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